

Cleaning of NETS Retrieval Systems and Equipment (re: COVID-19)

On Return to Base	Check
<ul style="list-style-type: none"> • Park ambulance in designated car space 	
<ul style="list-style-type: none"> • Open garage doors to create 'flow through' 	
<ul style="list-style-type: none"> • Returning or an available EVO to don PPE (contact + droplet) before removing system and equipment from vehicle – see chart 	
<ul style="list-style-type: none"> • Even, if in overtime, returning nurse/doctor in contact PPE to remove iStat and red drug pack, clean with Tuffies, doff PPE and return to Pharmacy Room 	
<ul style="list-style-type: none"> • If returning nurse and doctor are cleaning, grade up to PPE (contact + droplet). If they have already doffed PPE to take a refreshment break , don new PPE (contact + droplet) for cleaning 	
<ul style="list-style-type: none"> • If available on-base nurse and doctor are cleaning, don PPE (contact + droplet) 	
<ul style="list-style-type: none"> • Nurse/doctor in clean PPE to remove retrieval bags from vehicle and place on bench in garage clean-up area, before touching the contaminated system 	
<ul style="list-style-type: none"> • EVO in clean PPE to remove system from vehicle and place in space marked 'DIRTY' in clean-up zone. • If this is the returning EVO and he/she requires a refreshment break or is in overtime, doff PPE, discard in general waste bin in clean-up area 	
Cleaning of Systems and Equipment in Garage	
<ul style="list-style-type: none"> • Available EVO (in contact + droplet PPE) to deep clean ambulance as per policy • After cleaning ambulance, EVO to doff PPE, discard in general waste bin in clean-up area 	

Cleaning of NETS Retrieval Systems and Equipment (re: COVID-19)

Cleaning in Garage (continued)	Check
<ul style="list-style-type: none"> • Retrieval bags - wipe with Tuffies and allow to dry 	
<ul style="list-style-type: none"> • Linen – place in base linen skip 	
<ul style="list-style-type: none"> • Lead covers & harnesses – place in the empty tub for soaking and cleaning in the laundry 	
<ul style="list-style-type: none"> • Place all items that need to go to CSSD and/or Inhalation therapy in an appropriate size biohazard bag (separate bag for each department) 	
<ul style="list-style-type: none"> • Neonatal system canopy – remove, place on stainless steel bench and clean with Tuffies 	
<ul style="list-style-type: none"> • Systems, monitoring, pedimate, trolley/stretcher – clean with Tuffies, then move system to space marked ‘CLEAN’ 	
<ul style="list-style-type: none"> • When cleaning complete, take tub and biohazard bags to equipment room door 	
<ul style="list-style-type: none"> • Doff PPE, place in general waste bin in clean-up area. If PPE soiled, place in clinical waste bin. Don new gloves 	
In Equipment Restore Room	
<ul style="list-style-type: none"> • Dirty items in tub - take to laundry and follow cleaning instructions on wall 	
<ul style="list-style-type: none"> • Place pulse oximeter probe in Biomed basket 	
<ul style="list-style-type: none"> • Double bag CSSD/Inhalation items in biohazard bags as per photo instructions <ul style="list-style-type: none"> ○ grey heater wire in plastic box ○ place CSSD and Inhalation bags with forms in transport box 	
<ul style="list-style-type: none"> • Once systems and retrieval bags are dry (usually 1hr), take to restore room for replenishing 	